

# BrightView Family

## Cleaning the system components

### Plastic and Metal System Components

Moisten a soft, static-free cloth with a non-abrasive cleaning agent and wipe the exterior surfaces. While cleaning the plastic and metal system components, observe the following precautions:

- Do not spray aerosols or liquids on the imaging system equipment.
- Do not use solvents or abrasives on the imaging system equipment.
- Do not clean the internal components of the imaging system.



**CAUTION:** If a spill occurs and liquid enters the gantry, imaging table, or acquisition equipment, turn the system power off and contact Customer Support.

### Touchscreen Monitor Housing

Use a cloth lightly dampened with a mild detergent. While cleaning the touchscreen monitor housing, observe the following precautions:

- Do not use alcohol (methyl, ethyl, or isopropyl) or any strong solvent. Do not use thinner or benzene, abrasive cleaners, or compressed air.
- To avoid risk of electric shock:
  - Always unplug the monitor from the power outlet before cleaning.
  - Do not disassemble the AC adaptor or monitor; this unit is not user-serviceable.

### Touchscreen

Apply window cleaner or glass cleaner on a clean, soft cloth and gently wipe the touchscreen. While cleaning the touchscreen monitor housing, observe the following precautions:

- Never apply the cleaner directly on the touchscreen.
- Do not wipe the screen with a cloth or sponge that could scratch the surface.
- Avoid getting liquids inside your touchscreen. If liquid does get inside, turn the system power off and contact Customer Support before you power on the system again.

### Flat Panel X-Ray Detector (FPD)

To clean the FPD, follow the directions for cleaning the touchscreen as described in the previous section.

### Imaging Table

Clean the imaging table using a disinfectant in accordance with your site's protocol.

**\*IMPORTANT\*** Philips recommends that you clean and disinfect the imaging table after every patient.

### Patient Comfort Accessories

Clean the patient comfort accessories using a disinfectant in accordance with your site's protocol.