



HeartStart monitor/defibrillator

Cleaning and Disinfecting Information

General Recommendations:

Adhere to infection control measures, as outlined by your hospital and health department policies, in order to avoid transmission of the COVID-19 virus. Recommendations from the World Health Organization (WHO) and the Centers for Disease Control and Prevention (CDC) include using negative pressure isolation rooms for patients, and gloves, gowns and masks for caregivers.

It is important to note that the recommendations are not meant to replace existing national guidelines or established hospital protocols and following published guidelines and medical literature.

For a complete list of cleaning instructions, please refer to the Instructions for Use for your device.

| Product | Mild soap and water | Chlorine bleach Sodium Hypochlorite (Chlorine Bleach 3% Solution in water) | Chlorine bleach Containing 6% sodium hypochlorite | Quaternary ammonium compounds 21% quaternary ammonium content in water | Ammonia-based cleaners 10% in water | Alcohol-based cleaners 70% Isopropyl alcohol in water |
|---|---------------------|--|--|---|--|--|
| HeartStart HS1 AED | ✓ | ✓ | ✗ | ✗ | ✓ | ✗ |
| HearStart OnSite AED | ✓ | ✓ | ✗ | ✗ | ✓ | ✗ |
| HeartStart Home AED | ✓ | ✓ | ✗ | ✗ | ✓ | ✗ |
| HeartStart FRx defibrillator | ✓ | ✓ | ✗ | ✗ | ✓ | ✓ |
| HeartStart FR2 defibrillator | ✓ | ✓ | ✗ | ✗ | ✓ | ✓ |
| HeartStart FR3 defibrillator | ✓ | ✓ | ✗ | ✗ | ✓ | ✓ |
| HeartStart XL monitor/defibrillator | ✓ | ✓ | ✗ | ✗ | ✗ | ✓ |
| HeartStart XL+ monitor/defibrillator | ✓ | ✓ | ✓ | ✗ | ✗ | ✓ |
| Efficia DFM100 monitor/defibrillator | ✓ | ✓ | ✗ | ✗ | ✗ | ✓ |
| HeartStart Intrepid monitor/defibrillator | ✓ | ✓ | ✗ | ✗ | ✗ | ✓ |
| HeartStart MRx monitor/defibrillator | ✓ | ✓ | ✗ | ✓ | ✗ | ✓ |